



Setup Guide for SOTA Image 2.4.x or higher


- 1) Install the REALCLOUD HD1 camera button software utility. The installer is located on the REALCLOUD HD1 flash drive in the Button Software folder. Or REALCLOUD Website <http://www.realcloudimaging.com/support/>
Scroll down the Support Page to the Downloads Section. Click the “Capture Button Software” button. This will start downloading the Button-Tray-Utility.zip , once downloaded open the zip file and Install the REALCLOUD HD1 camera button tray utility software.
- 2) Select the Next button on the Welcome screen.

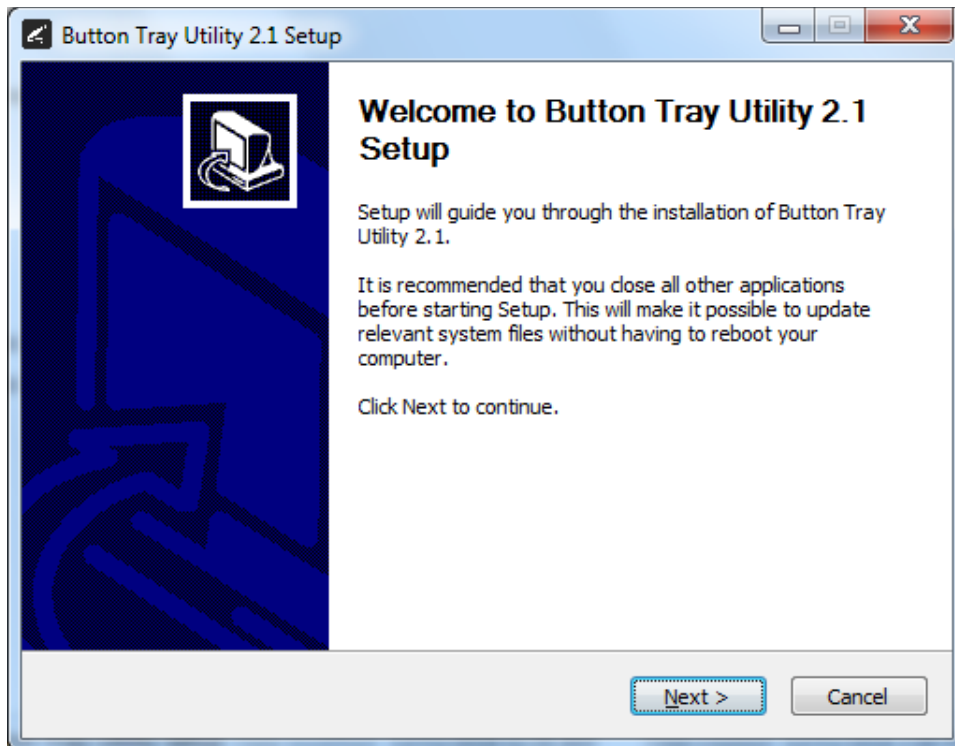


- 3) Select the install destination folder and click on the Install button.

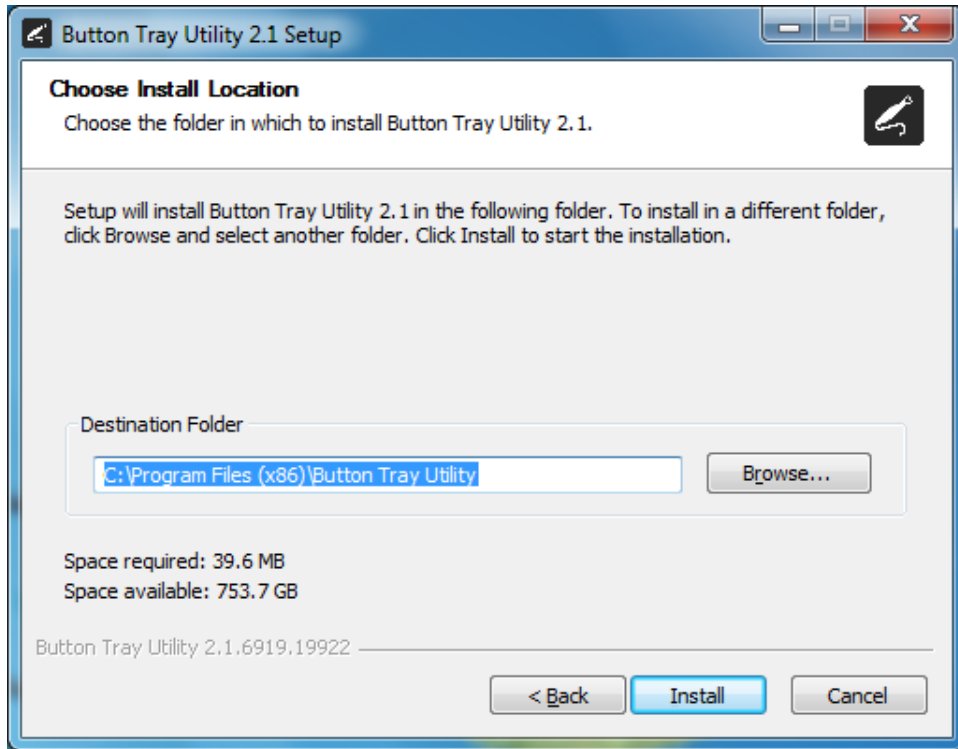


- 4) Plug the REALCLOUD HD1 camera into the computer and click on the OK button to start the installation process.

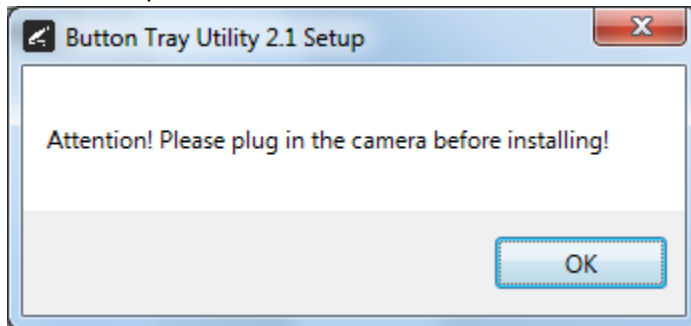


- 5) Click on the Next button to continue.

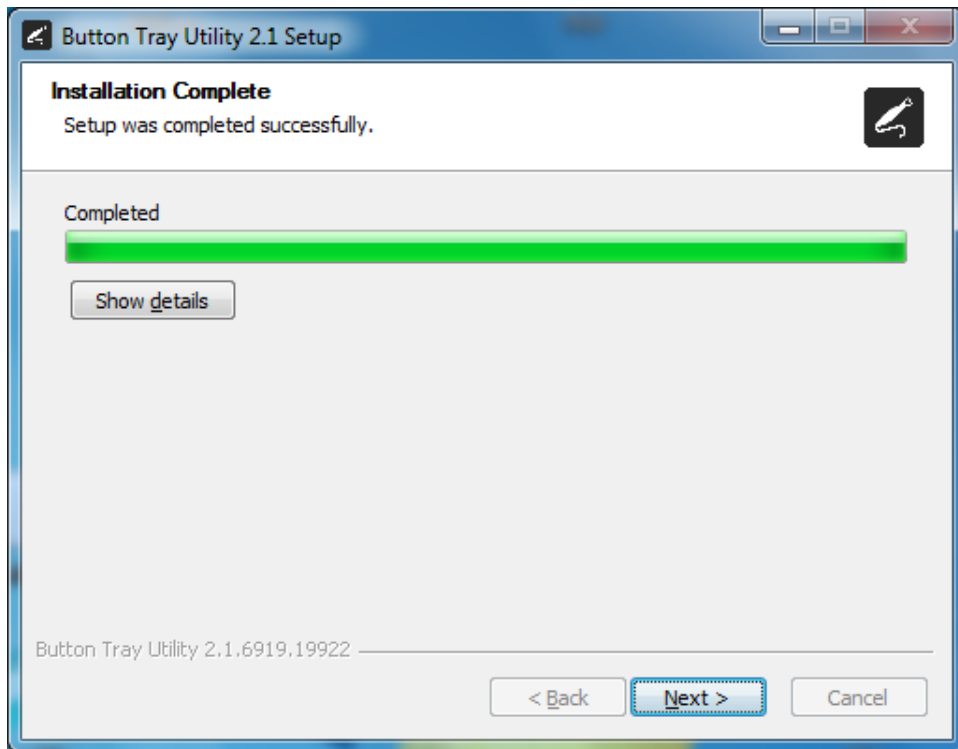


- 6) Select a reboot option (if prompted) and click on the Finish button.

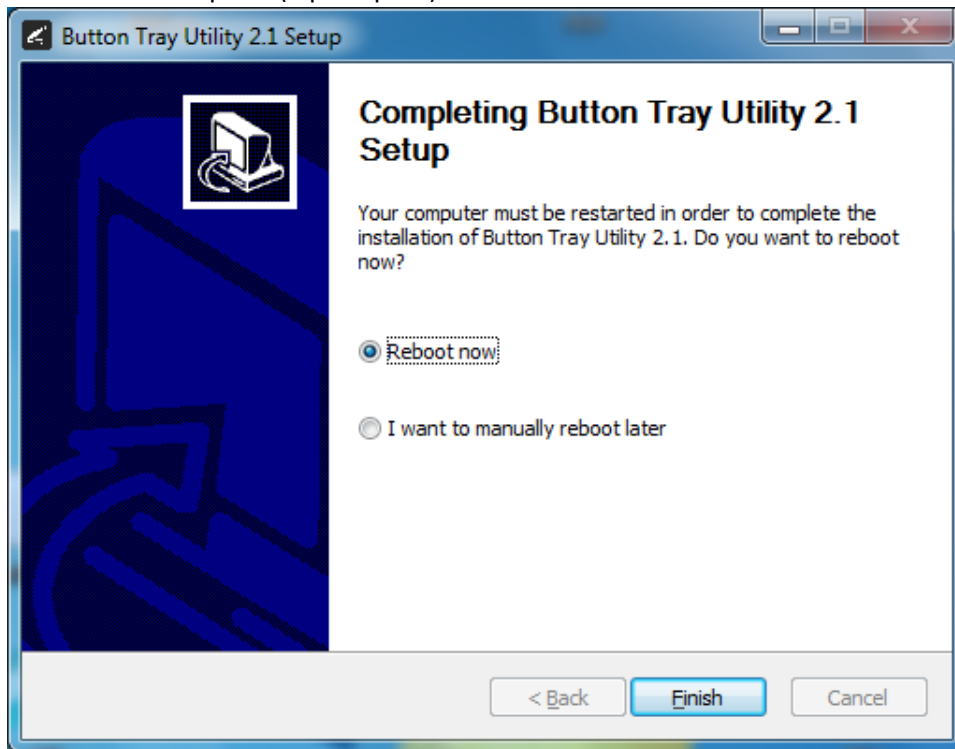



- 7) Right click on the Button Tray Utility icon in your system tray and select Preferences.

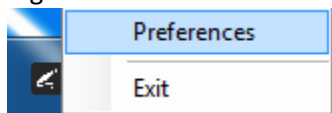


- 8) Click in the first text box for Short Press Keys and press the “c” key one time on your keyboard and in the second text box for Short Press Keys press the “s” key one time on your keyboard.
- 9) Click in the text box for Long Press Keys and press the “r” key one time on your keyboard, then click on the Done button.

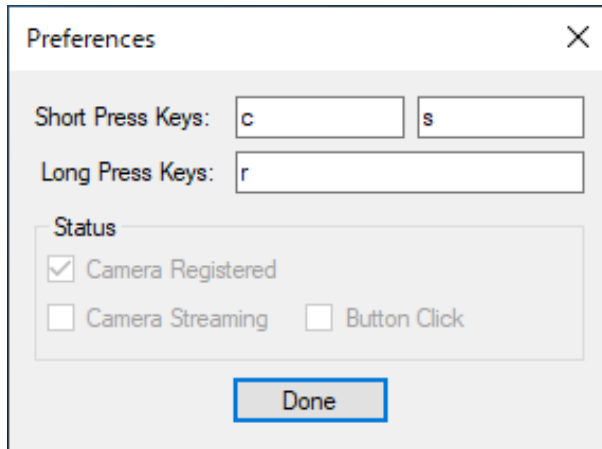


- 10) Contact SOTA Imaging technical support at 714-532-6100 to enable support for the REALCLOUD HD1 camera in the SOTA Imaging application.
- 11) Open SOTA Image application, go to Settings and select the Intraoral Camera button.

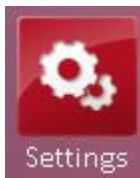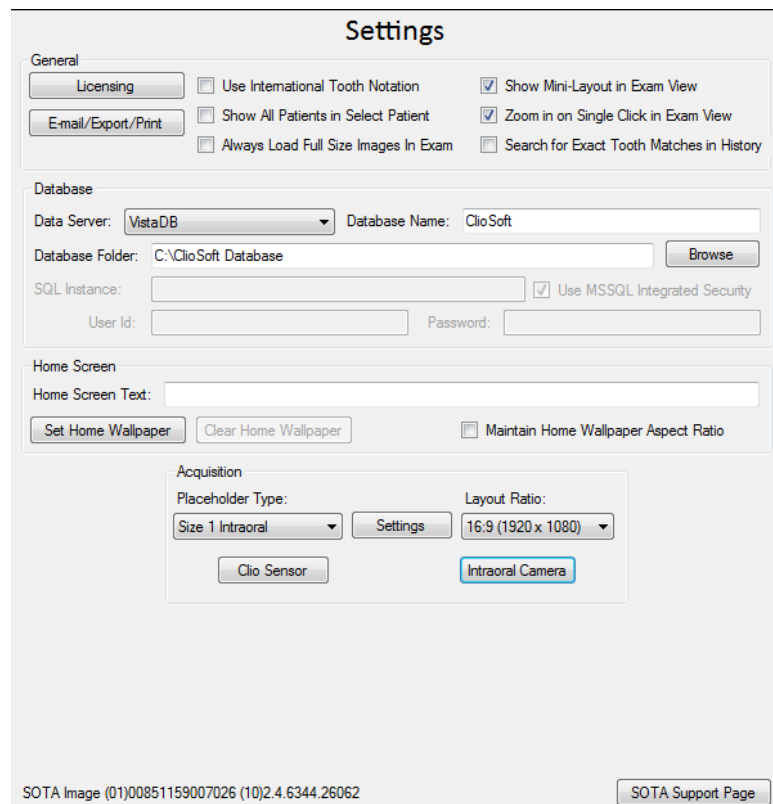


12) Ensure the “Use Advanced Video Rendering” option is selected and click the Done button.

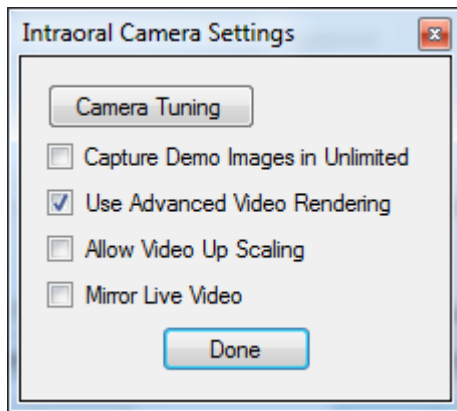

13) With a patient selected, click on the Camera button and select a Layout.

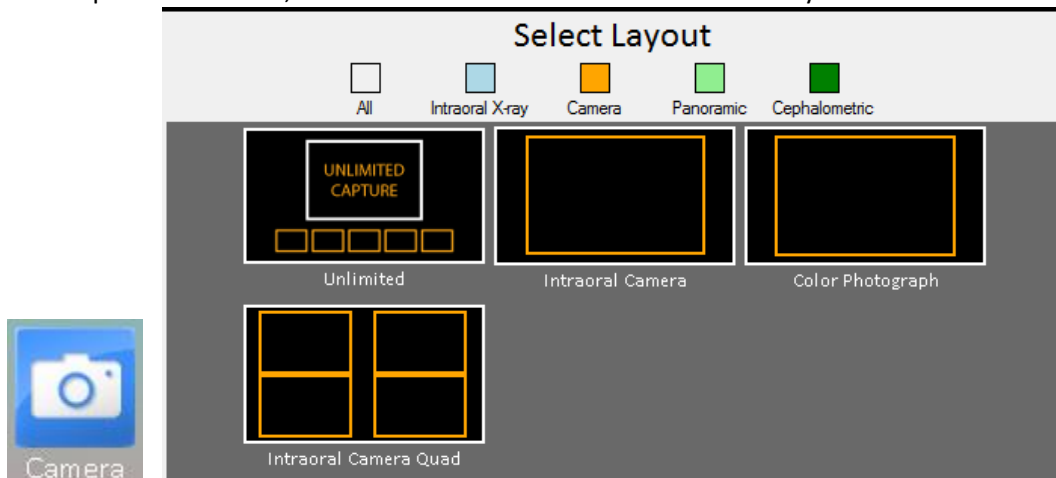

14) Click the Capture button to begin capturing image from the REALCLOUD HD1 camera.

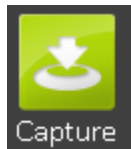

15) A short/quick camera button press, and release will pause video from the REALCLOUD HD1 camera. When a video image is paused, a short/quick camera button press will save the image. When a video image is paused, a long camera button press, hold (2 second), and release will return to live video without saving the image.