



Setup Guide for Visix 7.7.13 and higher.


- 1) Install the REALCLOUD HD1 camera button software utility. The installer is located on the REALCLOUD HD1 flash drive in the Button Software folder. Or REALCLOUD Website <http://www.realcloudimaging.com/support/>
Scroll down the Support Page to the Downloads Section. Click the “Capture Button Software” button. This will start downloading the Button-Tray-Utility.zip , once downloaded open the zip file and Install the REALCLOUD HD1 camera button tray utility software.
- 2) Select the Next button on the Welcome screen.

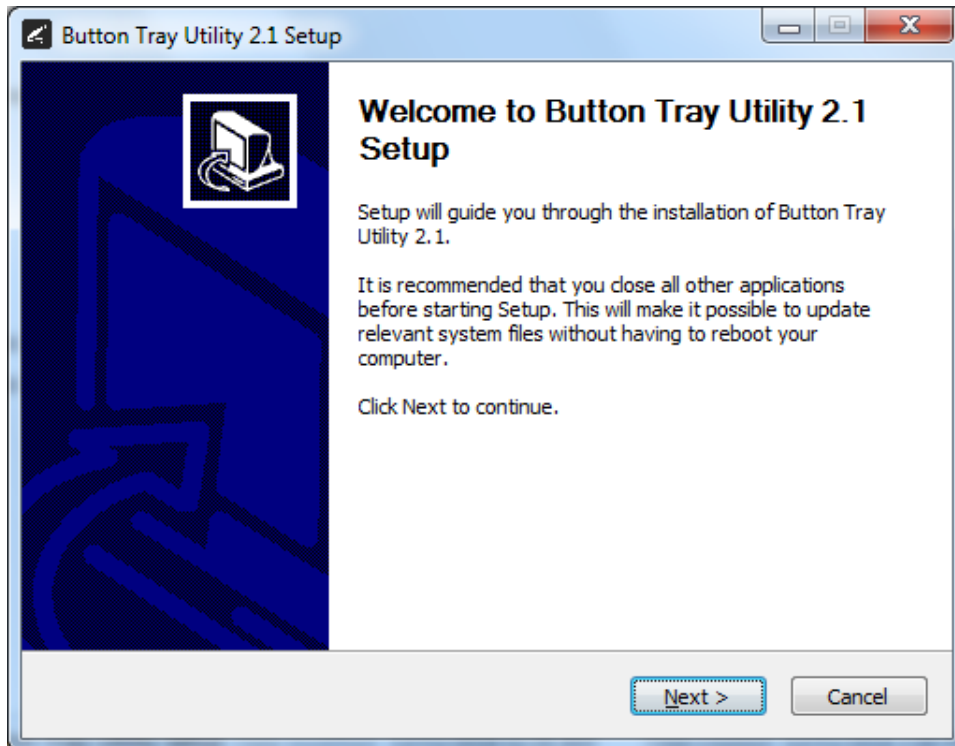


- 3) Select the install destination folder and click on the Install button.

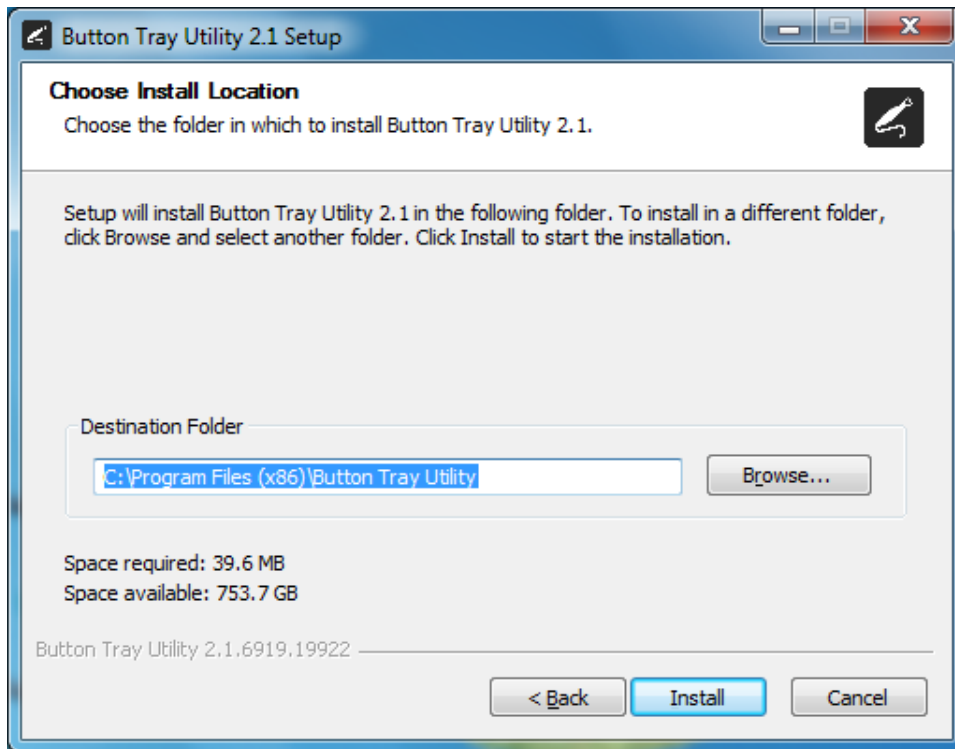


- 4) Plug the REALCLOUD HD1 camera into the computer and click on the OK button to start the installation process.

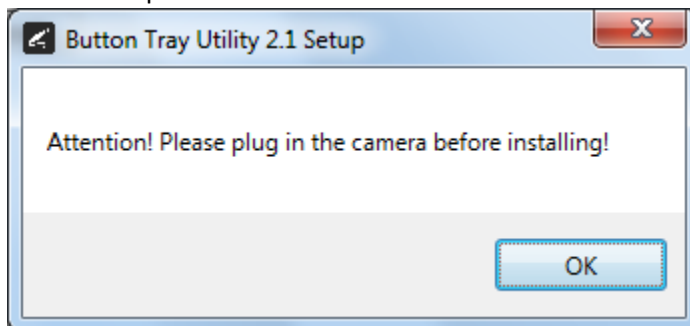


5) Click on the Next button to continue.

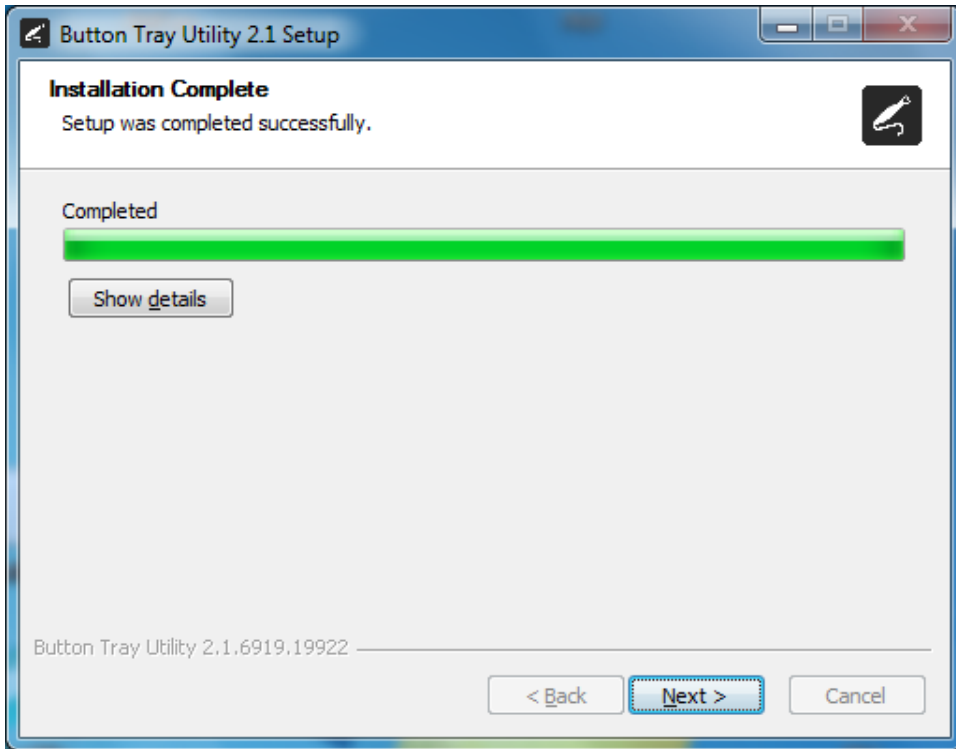


- 6) Select a reboot option (if prompted) and click on the Finish button.

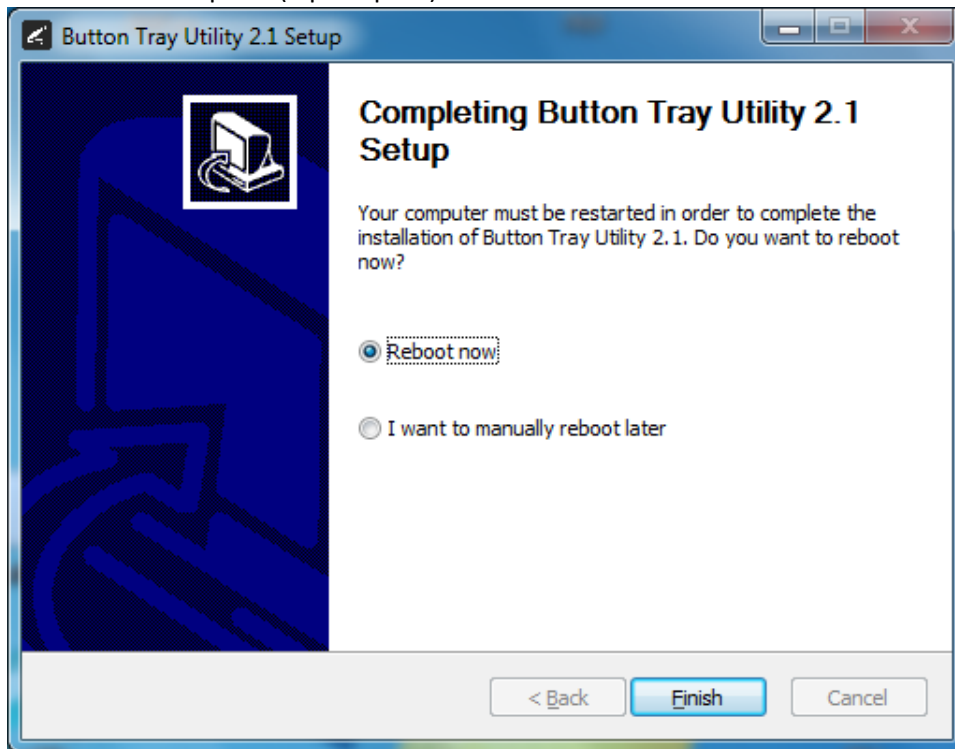

- 7) Right click on the Button Tray Utility icon in your system tray and select Preferences.

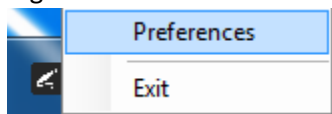

- 8) Click In the first text box for Short Press Keys and press the “f” key one time on your keyboard and in the second text box for Short Press Keys press the “s” key one time on your keyboard.
- 9) Click In the text box for Long Press Keys and press the “f” key one time on your keyboard, then click on the Done button.

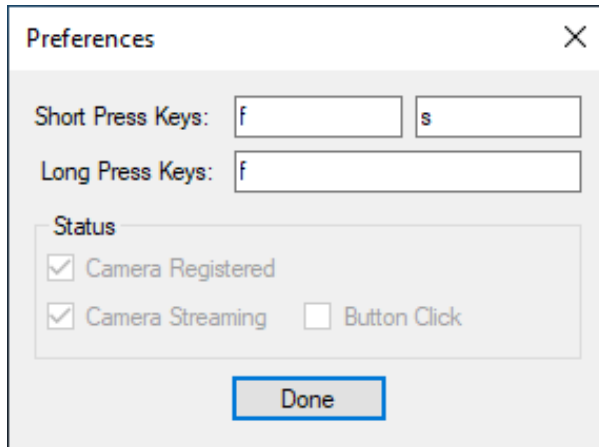

- 10) Open the Visix Administration application, select “Acquisition Hardware Configuration” and click the Next button.



11) Do not make any changes, click the Next button.



12) Select “Analog Video Camera or USB 2.0 Camera” and click the Next button.



13) Do not make any changes, click the Next button.




14) Select your preferred file format and click on the Next button.




15) Select your preferred startup acquisition mode and click on the Finish button.



16) Click on the Yes button.

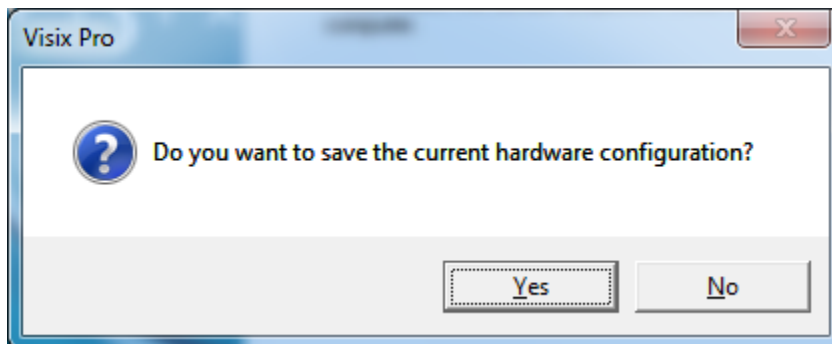


17) Click on the OK button.

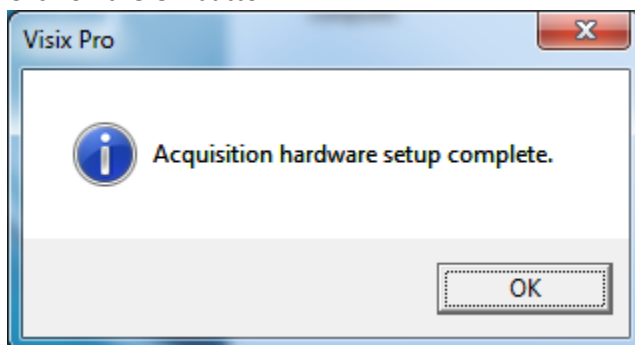


18) Click on the Close button.



19) With a patient selected click on the Acquire tab and click on the Video side tab.

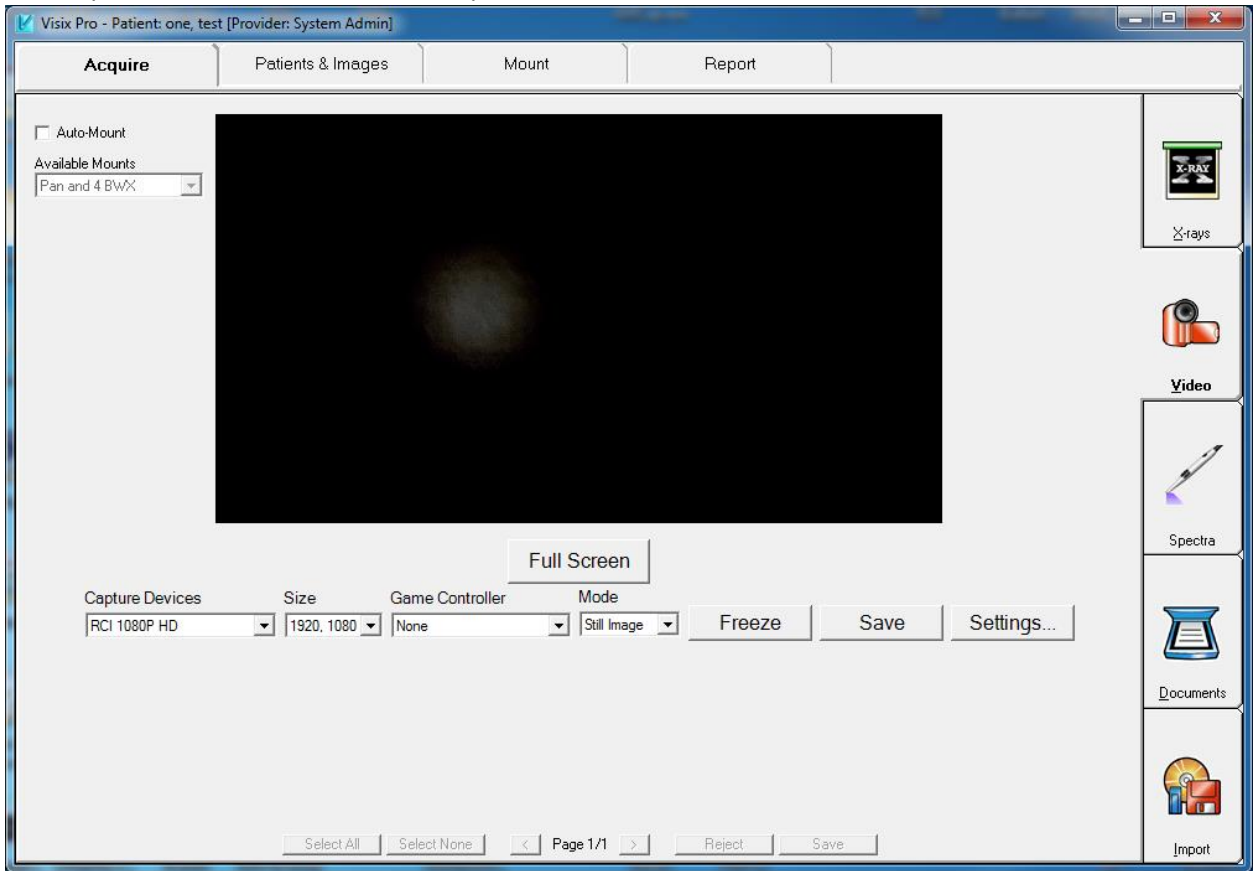

20) Click on the Settings button.

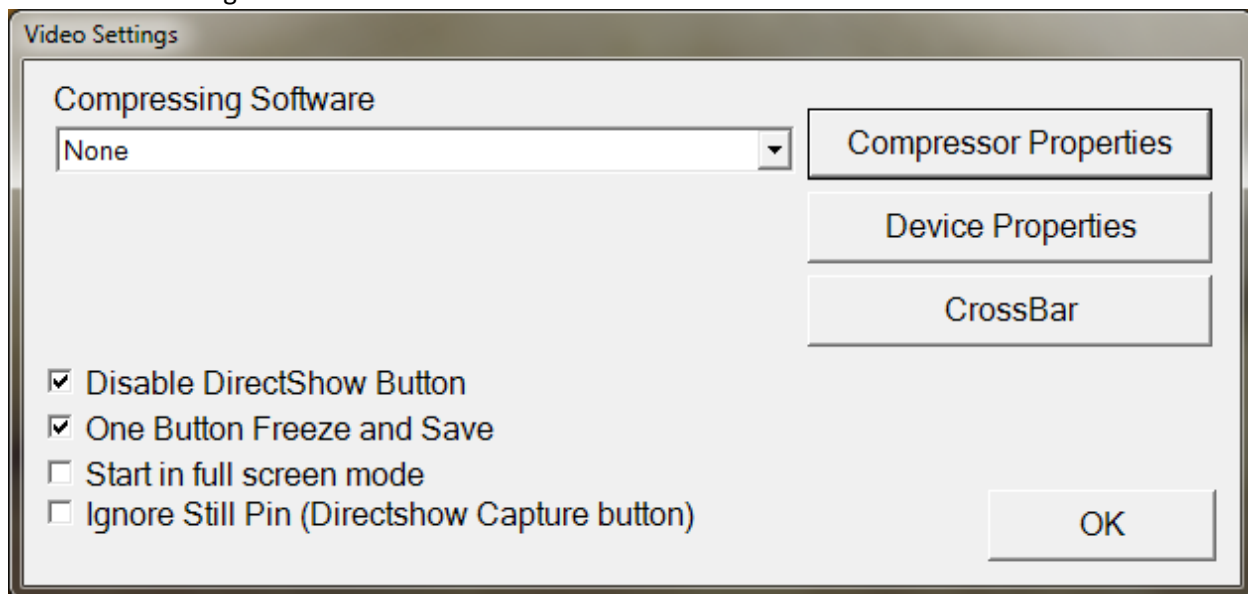

The Visix application can support the REALCLOUD HD1 camera button functionality in two different ways with or without “One Button Freeze and Save” option. Choose one of the following methods described below and click the OK button.

Without One Button Freeze and Save option selected: A short/quick camera button press, and release will pause the live video from the REALCLOUD HD1 camera. When the live video is paused on an image a second short/quick camera button press, and release will save the image and return to a live video. When the live video is paused on an image a long camera button press, hold (2 seconds), and release will return to live video without saving the image.

With One Button Freeze and Save option selected: The first short/quick camera button press, and release will pause the live video from the REALCLOUD HD1 camera. When the live video is paused on an image a long camera button press, hold (2 second), and release will save the image and return to a live video.